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**Thank you for choosing our product.** NOEL has designed and manufactured POSITIONING STABILO SYSTEM to the highest safety and functional use standards. We are confident that our product will meet your highest requirements and expectations. These instructions for use are an integral part of POSITIONING STABILO SYSTEM.

**Please read the instructions thoroughly before using the product.**

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**Vacuum positioning system stabilizing torso, head and limbs POSITIONING STABILO SYSTEM**

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Product designed for children and adults with cerebral palsy, compensating for and alleviating the disability by stabilizing the patient in sitting or lying position.

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**INTENDED USE**

POSITIONING STABILO SYSTEM can be used for:

- Alleviating the effects of remaining in one position for extended periods (e.g. contractures)
- Stabilizing patient with muscular flaccidity

**In the present instructions, the term cushion is used to refer to the STABILO SYSTEM.**

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## PRODUCT SAFETY WARNINGS

- POSITIONING STABILO SYSTEM must be securely fixed to seats or backrests
- Use of airy cover is advisable if increased ventilation is required around the body
- Avoid contact with sharp objects that may damage the cushion
- Do not clean using solvent-based cleaners, e.g. toluene or acetone
- Do not pull on valve
- Do not overinflate cushion
- Use only special pump included with cushion for inflation
- Do not iron
- In case of tear, micro-granules may escape which may be dangerous for small children

### ATTENTION

- The thickness of the cushion can naturally reduce by 5% in the initial period of use
- In case of considerable changes of temperature use pump again to obtain proper hardness (higher temperature softens the cushion, lower temperature hardens the cushion)

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## INSTRUCTIONS FOR USE

### Hand pump:

1. Take out tube in pump handle.
2. Screw tube into cushion valve and wait until cushion reaches the required softness.
3. Shape cushion (either manually to adjust shape or by sitting or leaning on it).
4. Screw pump into tube and deflate air until desired hardness has been reached.
5. Remove tube from pump and screw protecting cap on valve.
6. To change shape - screw tube into valve and wait until cushion softens, then repeat steps from 2 to 5.

### Electric pump

1. Take out tube from box with pump.
2. Screw off valve cap, screw tube (tapped part) into cushion valve, and push second tube into **SOFT** socket in electric pump. Turn on pump and wait until cushion will be soft enough, then turn pump off.
3. Start shaping cushion (either manually to adjust shape or by sitting or leaning on it).
4. Change position of tube from **SOFT** to **HARD** and turn pump on removing air until the desired hardness has been reached.
5. Screw off tube, put cap back on and tighten.
6. To change the shape repeat steps from 2 to 5.

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## MAINTENANCE

- Regularly check condition of product – pay attention to any cracks and rips
- Screw on valve cap before washing
- Hand wash cushion under running water using a standard detergent, e.g. a washing up liquid
- Suitable for washing in washing machine at 40°C cycle – fabric softener is sufficient, does not require use of washing powder
- Wipe dry with paper towel
- Standard disinfectants used in hospitals can be used
- Do not use solvent-based cleaners such as toluene or acetone
- For VELCRO products, Hoover the soft surface regularly to prevent loss of adherence.

## WARRANTY

- The warranty is provided for a period of **24 months**
- The warranty does not cover mechanical damage to POSITIONING STABILO SYSTEM
- If in doubt, please contact seller or manufacturer directly
- Report any incidents to seller or manufacturer directly

## DECLARATION OF CONFORMITY

POSITIONING STABILO SYSTEM medical device complies with the basic requirements of the Directive **EEC/93/42**.

**Fabriqué par :**

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**Distribué en France**

par :

**Vipamat**